



Instructions for use: Masking Powders

The shade required should be made from the 3 colours Ivory, Mustard and White by mixing in variable proportions with the white being the main constituent being more than opaque. A cold cure liquid should be used.

Having obtained a suitable shade the masking mix should be applied to retentions provided on the backing or metal structure so that the area is covered. When a satisfactory cover is obtained the work should be processed in a pressure flask before proceeding to the Acrylic processing.

Indications for use

For opaquing metal backings intended for use with acrylic facings for a chrome cobalt denture.

To mask and opaque metal structures and root canal posts for use with acrylic crowns and bridges