



Instructions for use: Metroclens

Uses in the dental laboratory and surgery.

1. **TRAY CLEANSER:** Use undiluted in an ultrasonic cleaner. Will remove most alginates and tray adhesives in 15 minutes. Old deposits of adhesive may require a further period to remove stubborn residues. Rinse well with clean water. Metroclens can also be used diluted 1:3 for overnight cleansing without ultrasonic assistance.
2. **PLASTER AND DENTAL STONE:** Use undiluted in an ultrasonic cleaner for best results. Will remove most deposits of plaster from between teeth in 15 minutes. Can be used for overnight soaking without ultrasonics. Rinse well with clean water.
3. **POLISHING COMPOUNDS:** Use up to 1:30 dilution depending on level of contamination. Ultrasonics will speed cleaning action. Rinse well in clean water.

4. **ZnO IMPRESSION PASTE:** Concentrated solution in an ultrasonic cleaner will remove ZnO paste from dentures, trays, instruments etc.

5. **BENCHES, CABINETS ETC.:** Dilute 1:100 for general cleaning of benches and cabinets etc.

No liability will be accepted for any direct or indirect loss or damage if the product is not used according to the directions and/or recommendations issued by the manufacturer.

CAUTION

Metroclens must not be used for aluminium or aluminium alloys