



## **Instructions for use: Supergel**

1. Mince Metrogel and place in gel tank container.
2. Heat up to approximately 90°C (DO NOT BOIL) until Metrogel becomes a smooth liquid.
3. Cool to 42°C stirring continuously to avoid lumps.
4. Soak model in warm water (26-28°C) for at least 5 minutes before fixing on base of flask.
5. Blow model dry, then pour Metrogel into flask until full and leave to set for about 90 minutes at room temperature.
6. Open flask, cut model free and lift out.
7. Wash the Metrogel after use. This serves the double purpose of replacing any of the water content which has been lost during use and ensures that the Metrogel is clean.
8. Keep material not in use in an airtight container.
9. When models are being duplicated in plaster, pour alum solution into mould, leave for five minutes, rinse and dry with compressed air.